ident

Innovations for Dentistry SA

-  **Gebrauchsanleitung für den EverClear 2 Dentalspiegel** (S.2)
-  **EverClear 2 Dental Mirror Operating Manual** (P.15)
-  **Mode d'emploi du miroir dentaire EverClear 2** (P.29)
-  **Manuale d'uso dello strumento di visione dentale EverClear 2** (P.43)

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Instructions for use for the EverClear 2 dental mirror

CONTENTS

1. Instructions for use
2. Technical description
 - 2.1 Technical data
3. Startup
 - 3.1 Contents of the set
 - 3.2 Connecting the power supply unit and the charger
 - 3.3 Recharging the batteries
4. Instructions for safe working
 - 4.1 The mirror change
5. Hygienic reprocessing and sterilization
 - 5.1 General notes
 - 5.2 Limitations and restrictions during reprocessing
 - 5.3 Pretreatment at the place of use
 - 5.4 Preparation prior to cleaning
 - 5.5 Manual cleaning & disinfection
 - 5.5.1 Instructions Batteries and other surfaces
 - 5.6 Machine cleaning & disinfection
 - 5.7 Drying
 - 5.8 Inspection, maintenance and care
 - 5.9 Packaging
 - 5.10 Sterilization
 - 5.11 Storage
6. Troubleshooting
7. Spare parts and consumables
8. Storage and transport
9. Explanation of the used symbols

1. INDICATIONS FOR USE

The EverClear 2 motorized dental mirror is intended for use by dental professionals and for use primarily in those dental applications where spray and drill dust may obstruct the view with conventional mirrors. To prevent personal injury or damage to property, please observe the warnings and safety instructions given in these instructions. They are specially marked with **CAUTION**, **ATTENTION** or **NOTE**.

NOTE: Indicates important information in the instructions to which you should pay attention.

CAUTION: Indicates the possibility of damage to the instrument.

ATTENTION: Indicates the possibility of injury to the patient or user.

This instrument must not be operated in potentially explosive atmospheres (e.g. with anesthetic gases).

The EverClear 2 instrument and charger are intended for use in a clean, indoor area of the dental practice and must be kept at room temperature between +10 C and +30 C.

Please read these instructions for use carefully before working with the instrument and keep them handy for reference.

Before using the instrument on a patient for the first time, it is recommended to disinfect and sterilize it according to the instructions in Chapter 5.

2. TECHNICAL DESCRIPTION

The I-DENT EverClear 2 complies with the regulations according to the current state of the art. According to the regulations, this product is to be used only for the described application, in compliance with the applicable hygiene regulations, occupational health and safety regulations, the applicable accident prevention measures and these operating instructions by the expert user. According to these regulations, it is the user's duty to use only fault-free work equipment, to ensure that it is used for the correct purpose, and to protect himself, the patient and third parties from danger.

2.1 Technical data

- Length of the handpiece: 170 mm, tapered
- Ergonomic handle: diameter 7.8 mm to 12.6 mm, non-slip, matt surface
- Handpiece weight: 60 g incl. batteries
- High quality material: medical stainless steel AISI 316L
- Durable precision ball bearings
- Interchangeable, two-sided precision mirrors:
 - Mirror diameter: 22 mm (No.4)
 - Optically high-quality, polished, stainless steel
 - Surface mirror (no double images)
 - Magnetically fixed and easily removable
- Direct drive by hermetically sealed brushless DC micromotor
- Electronically stabilized, controlled mirror speed (approx. 13,500 rpm)
- Long-life lithium-polymer batteries (LiPo), no memory effect, operating time of a fully charged battery: approx. 75 minutes
- Microprocessor controlled fast charger, input 5V, 400mA; time for a full charge: about 1 hour.

- Power supply: Please use a commercially available USB power supply unit (5V, 500mA minimum) in combination with the supplied cable
- The instrument is autoclavable: 135° C (= 275° F)

3. SET-UP INSTRUCTIONS

3.1 Contents of Box

#	Reference	Name	Starter Pack ID50-1900	Comfort Pack ID50-1901	Duo Pack ID50-1902
1	ID50-1913	USB-A to USB-C cable	1x	1x	1x
2	ID50-1911	Charging adapter	1x	-	-
3	ID50-1910	Charging station	-	1x	1x
4	ID50-1912	Battery pack	1x	2x	2x
5	ID50-1916	EverClear 2 instrument	1x	1x	2x
6	ID50-1915	Lubrication spray adapter	1x	1x	1x
7	ID50-1914	Mirror exchange tool	1x	1x	1x

Fig 1

3.2 Connecting the charging station and the charging adapter

Snap the plug of the power supply cable (1) into the corresponding socket on the back of the charging adapter (2) or the charging station (3) (Fig. 2 & 3). The blue indicator lamp shows that the charger is supplied with power.

Fig 2

Fig 3

CAUTION: Please position the device with a minimum distance to the patient of 1.5 m on a stable, horizontal surface.

3.3 Recharging the batteries

Please fully charge the batteries before using them for the first time. To do this, place the battery (4) on the connection adapter and push it in until the adapter clicks into place (Fig. 5). The red control lamp lights up to indicate that the charging process has been started.

Fig 4

Fig 5

When charging is complete, the red indicator lamp goes out; the blue indicator lamp lights up. When the instrument is not in use, the batteries should be left on the charger for storage to compensate for self-discharge and to ensure the longest possible battery life. These batteries do not exhibit memory effect and should always be kept on the charger whenever possible. Fully discharged batteries take a little over an hour to fully recharge. With a charged battery, the EverClear 2 instrument can be operated continuously for approximately 75 min. As is common with all battery-powered instruments, battery capacity will slowly decrease with age, reducing operating time. Battery replacement is then at the discretion of the user.

NOTE : These batteries may only be charged with the I-DENT charger provided for this purpose.

NOTE : EverClear 2 batteries are not compatible with EverClear 1 chargers

NOTE : Do not insert metal parts into the open end of the battery; this could cause the battery to short circuit and heat up.

CAUTION: Never autoclave the batteries!

4. INSTRUCTIONS FOR SAFE OPERATION

Fig 6

Fig 7

Gently insert the battery (4) into the EverClear 2 handpiece (3). You will feel,

how the connector plug is automatically guided to the OFF position (Fig. 6). When you feel the resistance, push the battery fully into the handpiece (Figure 7); it will engage with a "click". To start the EverClear 2 instrument, rotate the battery a quarter turn in any direction. To stop the EverClear 2, turn the battery again a quarter turn in any direction.

When in the **ON** position, the mirror will rotate at high speed to expel water and drill dust. EverClear 2 improves the efficiency of most dental procedures by keeping the mirror surface free of water droplets and drill dust, eliminating the need to constantly remove the mirror from the mouth for cleaning.

When processing some composite materials, the electrostatic properties of these materials can cause individual particles to adhere to the surface, resulting in easily visible circles on the mirror. Any adhesions can be removed immediately by briefly wiping the mirror with the instrument spray.

Visibility can be additionally optimized by slightly increasing the amount of water spray on the handpiece.

In principle, EverClear 2 reduces the need to decrease the amount of spray, which is often used by experienced dentists to maintain visibility for as long as possible. The use of higher spray volumes improves cooling of the tooth and is therefore a significant advantage of EverClear 2. If the mirror surface comes into contact with an object, rotation is slowed or stopped altogether. Although contact with intraoral tissue is absolutely safe, you should avoid any contact of the rotating mirror surface with cannula, soft tissue or handpiece to maintain continuous function during treatment. If rotation stops due to contact, it will resume shortly after the mirror surface has detached from the contacting object.

CAUTION: To avoid scratching the mirror surface during preparation, be sure to position the mirror slightly away from the head of the burr instrument. Likewise, avoid any contact of the EverClear 2 with rotating drills to prevent damage to the surface of the mirror or the instrument itself.

NOTE: It is normal for the head of the EverClear 2 to heat up slightly during treatment. It will feel warmer if rotation is impeded by touching the mirror, as described above.

When the treatment is finished or interrupted, switch EverClear 2 to **OFF** to stop the rotation. In the event that a battery is discharged during a treatment, EverClear 2 will stop rotation; the discharged battery must be replaced with the second, charged battery.

NOTE : Due to the electronic speed stabilization, there is no noticeable loss of speed when the battery charge decreases. Speed is maintained until the mirror stops due to complete discharge of the battery.

NOTE : For proper operation, keep battery contacts clean and dry.

4.1 Mirror exchange

To remove a mirror from the support plate to replace the scratched mirror, for cleaning or maintenance, first remove the rotor by pressing on the rear of the rotor (Fig. 8) until the rotor is released from its fixing with a "click" (Fig. 9).

Place the rotor in the mirror exchange tool so that the recesses on the rotor are aligned with the recesses of the tool.

Use two fingers to push the mirror straight down until the magnetic attraction force is overcome. The rotor remains in the tool.

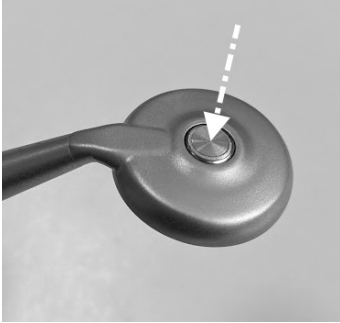
Fig 8

Fig 9

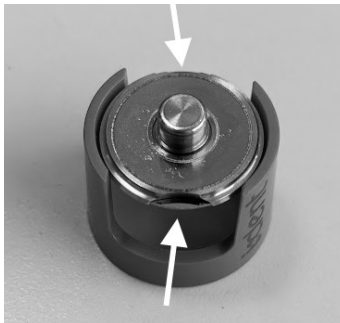
Fig 10

Fig 11

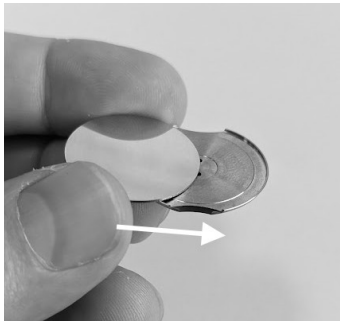
Fig 12

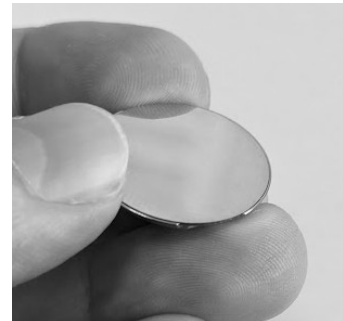
Fig 13

Fig 14

Fig 15

Position the mirror at an angle on the mirror support (Fig. 12). Slide the mirror sideways until it touches the circumferential edge of the mirror support plate and set it down at the same time.

The mirror will insert itself within the circumferential edge of the centering seat as it is lowered onto the mirror support. Make sure the mirror is centered and completely evenly inserted into its mount (Figure 13).

To reinsert the rotor into the head housing, position the rotor in the holder (Fig 14) and engage it with slight axial pressure (Fig 15).

ATTENTION: After each mirror replacement, please activate the motor to ensure that there is no vibration and that the mirror is properly inserted. Check EverClear 2 before inserting it on the patient. Incorrect insertion of the mirror can cause the mirror pane to detach from the rotor when the instrument is switched on.

To ensure the best viewing quality after the second side of a mirror has also been damaged, please use a new replacement mirror. Refer to Chapter 9 "Spare parts and consumables" for ordering instructions.

5. HYGIENIC REPROCESSING AND STERILIZATION

5.1 General notes

All instruments must be cleaned and disinfected before each use; this applies in particular to the first use after delivery. For the first cleaning and disinfection, the medical device must be removed from the transport packaging.

Sterilization is not necessary due to the classification of the instruments as semicritical, but can be performed to obtain a completely germ-free instrument.

Effective cleaning and disinfection is an indispensable prerequisite for effective sterilization.

In principle, only device- and product-specific validated procedures for cleaning, disinfection and sterilization may be used.

- The devices used (washer/disinfector, sterilizer) must be regularly maintained and checked.

Also observe the legal regulations applicable in your country as well as the hygiene regulations of the medical practice or hospital.

ATTENTION: During all steps of preparation (the collection, pre-cleaning, intermediate storage and transport), occupational health and safety concerns, e.g., suitable protective clothing, protective goggles, suitable gloves must be ensured (TRBA 250). I-dent has validated the following processing instructions as suitable for reprocessing a medical device for the purpose of reuse. The processor is responsible for ensuring that the reprocessing actually carried out achieves the desired results with the equipment, materials and personnel used at the reprocessing facility. This usually requires validation and routine monitoring of the process. Deviations from the specified instructions by the reprocessor must be carefully evaluated for effectiveness and potential adverse consequences.

ATTENTION: I-dent recommends that only EN ISO 15883-1 and -2 compliant washer-disinfectors be used for machine cleaning/disinfection. According to the RKI recommendation, machine reprocessing should be given preference over a manual reprocessing method, if possible. A manual method should only be used if a machine method is not available. In this respect, manual reprocessing is not intended for the product.

5.2 Limitations and restrictions during reprocessing

CAUTION: When selecting cleaning and disinfecting agents, always make sure that they do not contain any of the following ingredients:

- Organic, mineral, or oxidizing acids (minimum allowable pH 5.5)
- Strong alkalis (maximum permissible pH 11; we recommend a neutral / enzymatic or slightly alkaline cleaner)
- Organic solvents (e.g. ether, ketone, gasoline)
- Oxidizing agents (e.g. hydrogen peroxides)
- Halogens (chlorine, iodine, bromine)
- Aromatic / halogenated hydrocarbons
- Heavy metal salts

NOTE: Only use cleaning/disinfecting agents that are approved for reprocessing medical devices. Observe the manufacturer's instructions for the cleaning agents/disinfectants.

If unsuitable cleaning agents or disinfectants are used or unsuitable cleaning or disinfection procedures are applied, this may have negative consequences for the instruments:

- Damage or corrosion
- Discoloration of the product
- Corrosion of metal parts
- Shortened service life

CAUTION : Never clean medical devices with metal brushes or steel wool.

CAUTION : All instruments must only be exposed to temperatures below 138 °C (280 °F; plus tolerance according to DIN EN ISO 17665-1:2006)!

The end of life (time at which reprocessing can no longer be considered safe) is mainly determined by mechanical stress during the use of the instruments and chemical influence due to processing. Defective instruments, e.g. with corroded surfaces, must be disposed of by the operator and must not be used any further.

CAUTION: Do not place the EverClear 2 in the disinfection solution. Do not use an ultrasonic bath for reprocessing!

NOTE : The instrument parts rotor, mirror surfaces and holder sing suitable to be sterilized at up to 135°C (275°F).

CAUTION : The battery and mirror changer are not suitable for sterilization.

5.3 Pretreatment at the point of use

Pre-cleaning of the instruments should be performed immediately after treatment (within a maximum of 2 h), taking personal protection into account. The aim is to prevent organic material and chemical residues from drying in the lumen or on the outer parts of the instruments and to avoid contamination of the environment.

Use running tap water for preliminary cleaning. Only use a soft brush (e.g. nylon brush/toothbrush) for manual removal of contamination, and never use metal brushes or steel wool for this purpose. Make sure that you have covered all surfaces and that no visible residue remains.

To avoid damage, the instruments should be stored safely and transported in a closed container (e.g., a tub with a lid) to prevent contamination of the environment to the place of further reprocessing.

CAUTION : Do not place the EverClear 2 in cleaning or disinfecting solution.

5.4 Preparation prior to cleaning

The instrument must be disassembled as described in 4.1. All cleaning and disinfecting agents should be used with the dilution, temperature and exposure time recommended by the manufacturer.

5.5 Manual cleaning & disinfection

Manual reprocessing is not intended for the rotor, mirror surfaces and handpiece.

NOTE: Observe your local and national laws, guidelines, standards and specifications for cleaning disinfection and sterilization.

CAUTION: Wear protective clothing, goggles, protective mask and gloves.

WARNING: Do not place the EverClear 2 in the disinfection solution or ultrasonic bath!

5.5.1 Instructions Batteries and other surfaces

The outside of the battery can be cleaned and disinfected by wiping disinfection. Use approved alcohol-based instrument disinfectants for this purpose.

ATTENTION : The battery contacts must remain dry.

The surface of the charger can also be cleaned and disinfected with a mild detergent or alcohol-based disinfectant.

The same applies to the mirror changer. Always use only soft, lint-free cloths for the plastic surfaces to avoid scratches.

5.6 Mechanical cleaning & disinfection

CAUTION: Remove the rechargeable battery before machine cleaning. The rechargeable battery must be reprocessed as explained in 5.5.2 and is not suitable for machine reprocessing.

When selecting the disinfection device, make sure that the disinfectant has proven effectiveness (e.g. DGHM or FDA approval or CE marking according to DIN EN ISO 15883) and, if possible, a tested program for thermal disinfection (A0 value > 3000 or - for older devices - at least 5 min at 90° C/ 194° F) is used.

I-dent recommends the use of an EN ISO 15883-1 and -2 compliant washer-disinfectant in combination with a suitable loading carrier (e.g. associated sieve basket). Follow the operating instructions of the device manufacturer of the washer/disinfectant.

Load the washer/disinfectant according to the manufacturer's specifications.

NOTE: Observe your local and national laws, guidelines, standards and specifications for cleaning disinfection and sterilization.

CAUTION: Wear protective clothing, goggles, protective mask and gloves.

Procedure:

Mechanical cleaning and disinfection

1. Place the rotor and the mirror element separately in a suitable small parts insert. The rotor should be placed with the mirror contact surface facing downwards (or vertically). Make sure that the instruments do not touch each other.

CAUTION : The handpiece must be placed vertically in the WD to ensure that moisture does not remain permanently inside the handpiece. This can lead to corrosion of the internal battery contacts.

2. Start the program.

3. Remove the instruments from the WD at the end of the program.

4. Check and pack the instruments as soon as possible after they have been removed (see "Checking, maintenance and care" section, if necessary after additional drying in a clean place).

Proof of the basic suitability of the products/product components for effective manual cleaning and disinfection was provided by an independent accredited and recognized (§ 15 (5) MPG) test laboratory (HygCen Germany GmbH, Schwerin). The reprocessing was validated under the following conditions in a washer-disinfectant type Miele Professional PG8581 with the cleaner neodisher® Mediclean Dental (0.5%, Fa. Dr. Weigert) in the Vario TD program:

- Step 1 Pre-cleaning with cold tap water <40°C for 1 minute.
- Step 2 Cleaning 0.5% cleaner 55°C for 5 minutes with demineralized water
- Step 3 Rinsing with demineralized water for 1 minute
- Step 4 Thermal disinfection with demineralized water >90°C for 5 minutes

After thermal disinfection, a drying step can be included in the program of the washer/disinfectant.

CAUTION: Never store the instruments wet after reprocessing. This can lead to corrosion and germ growth. If the instruments are not completely dry after completion of the machine reprocessing, re-dry them manually (see point Drying) and then store the instruments accordingly.

NOTE: Remember that every cleaning and disinfection process should be validated.

NOTE: The validated parameters correspond to a process with an A0 value of >3000s. I-dent recommends using only processes that meet or exceed this value.

NOTE: Before possible sterilization, relubrication should now be performed as described in chapter 6.

5.7 Drying

Dry the instruments with medical compressed air (oil-free, low in germs and particles) using a reprocessable compressed air gun, air pressure: max. 3 bar, room temperature or a clean, absorbent and lint-free cloth. Make sure that the handle of the handpiece is also dry from the inside before reinserting the battery.

5.8 Control, maintenance and care

The EverClear 2 ball bearings require regular lubrication. I-DENT recommends oiling before each sterilization process. Commercially available turbine oil can be used for the EverClear 2.

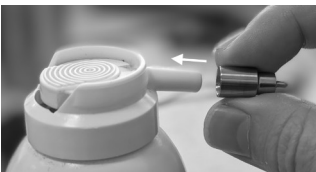
Fig 16

Preparing the spray bottle: Before first use, please insert the large end of the supplied spray adapter (6) over the spray head of your spray can up to the stop, approx. 1cm deep (Fig. 16).

To relubricate, remove the mirror from the rotor using the mirror changer (as described in 4.1). Insert the fine nozzle end of the spray bottle into one of the two openings in the center of the rotor as far as it will go and press the button on the spray bottle (Fig. 17). A short burst of spray is sufficient.

Fig 17

Carefully wipe off any excess lubricant from the surface of the rotor. Place the mirror back on the rotor and run EverClear 2 for one minute.

As with all dental instruments, cleanliness is of paramount importance.

In particular, make sure that the mirror support disks under the mirror surface are clean and free of particles. All EverClear 2 surfaces, charger, mirror changer, handpiece, batteries and the mirror can be cleaned with detergents, alcohol or alcohol-based disinfectants. Dampen a clean, lint-free cloth with disinfectant and thoroughly wipe the exterior surfaces.

The battery contacts do not require cleaning and must remain dry.

Place the instrument, rotor, and mirror element without battery in a sterilization bag and autoclave according to the sterilizer manufacturer's instructions for use. The maximum nominal temperature for sterilization is 135 °C (275 °F).

ATTENTION: After each replacement of the mirrors, please activate the motor to make sure again that the mirror is centered and completely evenly inserted in its holder. The wrong position of the mirror could cause the mirror to detach from the rotor, as noted in chapter 4.1.

5.9 Packing

Not applicable.

5.10 Sterilization

NOTE: Do not sterilize the mirror changer!

NOTE: Do not autoclave the batteries under any circumstances!

NOTE: Do not forget regular lubrication as described in Chapter 6

CAUTION: Do not use hot air sterilization, radiation sterilization, formaldehyde or ethylene oxide sterilization, chemiclaves or plasma sterilization.

NOTE : I-dent does not provide for sterilization due to its classification as a semicritical instrument. The rotor, mirror surfaces and handpiece are basically suitable for sterilization. Place the handpiece, the rotor, and the mirror element, **without the battery**, in suitable sterilization packaging. Follow the sterilizer manufacturer's instructions for use when sterilizing. The maximum temperature of the sterilization process should not exceed 135°C (=275°F).

5.11 Storage

After sterilization, the products/product components must be stored in the sterilization packaging in a dry and dust-free place.

8. TROUBLESHOOTING

In rare cases, the rotation may not restart after it has stopped once. This indicates increased friction in the bearing of the rotor. This can be caused either by the consumption of lubricant after long use, or by deposits in the bearing that can enter during autoclaving due to contaminated water vapor atmosphere. First aid: Place your finger on the mirror surface and rotate the mirror counterclockwise.

The rotation will start again. At the next maintenance or before the next use, perform a relubrication as described above. If relubrication does not correct the problem, deposits are probably the cause and the instrument should be returned to your dealer for service.

CAUTION : Batteries should never be autoclaved. Autoclaving the EverClear 2 with the battery may cause damage to the instrument.


Accidental sterilization of the battery or unintentional rapid discharge by a metallic object may cause a small amount of battery fluid to leak from the contact side of the battery. This is aggressive and must therefore be removed immediately. To do this, wash all parts and your hands carefully under running water. The battery in question must then be disposed of in accordance with regional regulations.

If you require further assistance, please contact your dealer or write an e-mail to I-DENT Innovations for Dentistry: info@i-dent-dental.com

9. SPARE PARTS AND CONSUMABLES

Please always use only approved or original spare parts:	Order number
Double-sided precision surface mirrors (6 pieces/12 usable surfaces)	ID50-2070
Battery pack (Metal Encapsulated LiPo Battery with Quick Disconnect)	ID50-1912
Spray adapter for lubrication	ID50-1906
Mirror changer (mirror removal tool)	ID50-1914
Replacement rotor	ID50-1918

10. STORAGE AND TRANSPORT

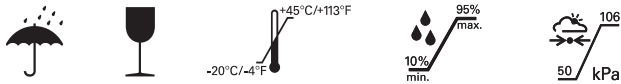11. EXPLANATION OF THE SYMBOLS USED

The EverClear 2 instrument bears the CE mark in accordance with the provisions of Directive 93/42/EEC.

This symbol indicates that the EverClear 2 instrument can be autoclaved at a temperature of 135°C (= 275°F).

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Please observe the accompanying documents!

Date of manufacture

Medical device

Can be thermally disinfected

Article reference number

Serial number

This symbol indicates separate collection of electrical and electronic equipment in EU countries. Please do not dispose of the device in household waste. Find out about the take-back system applicable in your country from your local waste disposal company or your dealer and use this for disposal.

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