Route de Champ-Colin 20
CH-1260 Nyon
Switzerland
Tel. : +41 22 525 31 80
e-mail info@i-dent-dental.com
web www.i-dent-dental.com

iident

Innovations for Dentistry SA

-  **Gebrauchsanleitung für den EverClear™ Dentalspiegel** (S.2)
-  **EverClear™ Dental Mirror Operating Manual** (P.13)
-  **Mode d'emploi du miroir dentaire EverClear™** (P.24)
-  **Manuale d'uso dello strumento di visione dentale EverClear™** (P.36)

INHALT

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CONTENTS

1. Indications for use
2. Technical description
 - 2.1 Technical data
3. Set-up Instructions
 - 3.1 Contents of box
 - 3.2 Connecting the power supply to the battery charger
 - 3.3 Charging batteries
4. Instruction for safe operation
 - 4.1 Mirror exchange
5. Hygienic reconditioning and sterilization
 - 5.1 Removal of the Rotor assembly
 - 5.2 Battery and other surfaces
6. Maintenance
7. General care
8. Troubleshooting
9. Spare parts and consumables
10. Storage and transport conditions
11. Explanation of symbols used on the device

1. INDICATIONS FOR USE

The EverClear™ motorized dental mirror is intended for use by dental professionals in various dental procedures in which spray and debris may obstruct the view in a regular dental mirror. Please observe the warnings and safety instructions - highlighted in this operating manual by the captions

ATTENTION, CAUTION and NOTE - to avoid the possibility of personal injury or material damage.

NOTE: indicates a particular point of information, to focus on.

CAUTION: indicates the possibility of damage to the device.

ATTENTION: indicates the possibility of injury to the patient or the user.

This product must not be used in areas subject to explosion hazards.

The EverClear™ instrument and charging device are intended for use in clean, interior areas of a dental practice and at room temperatures ranging from +10°C to +30°C.

Read these operating instructions carefully prior to operating this instrument and keep them handy for further reference.

Prior to any use with a patient, instrument disinfection and sterilization are recommended according to the instructions in chapter 5.

2. TECHNICAL DESCRIPTION

The EverClear™ dental mirror complies with the latest regulations reflecting the current state of the art. According to the regulations, this product may only be used for the described applications by trained personnel in compliance with the applicable hygiene standards, occupational safety regulations and accident prevention measures as well as these operating instructions. According to these regulations, the user is instructed to use only equipment in good working order, to pay attention to correct applications as well as to protect him- or herself, the patient, and third parties against hazards.

2.1 Technical data

- Handle length: 170 mm, tapered
- Ergonomic grip: diameter 7.8mm to 12.6mm, with non-slip, matt finish
- Handpiece weight: 64g including battery
- All high-grade materials: medical stainless steel AISI 316 L, titanium and ceramic
- Precision ball bearings, ceramic hybrid, long lasting
- Replaceable, double-sided precision mirrors:
 - o mirror diameter 22 mm (# 4)
 - o high-grade optically polished stainless steel
 - o front surface reflecting,
 - o magnetically retained and easily removable
- Hermetically sealed, brushless, DC direct-drive micro-motor
- Electronically stabilized, controlled mirror speed (14,000 rpm)
- Nickel metal hydride (NiMH) long lasting batteries, no memory effect, useful life per full charge approx. 45 min.
- Microprocessor controlled quick charge type battery charger, input 6 V, 800mA; Full-charge time approx. 1 hour
- Charger capacity: two batteries simultaneously

- Power supply: small plug type, 100 to 240 V AC, 60/50 Hz, 7.2VA, North American, United Kingdom and European plugs available
- Autoclavable: 135° C (275° F)

3. SET-UP INSTRUCTIONS

3.1 Contents of box

- 1 – Cord with Power Supply
- 2 – Battery Charger
- 3 – EverClear™ Instrument
- 4 – Rechargeable Batteries (two)
- 5 – Magnetic Mirror Exchange Tool
- 6 – Double Sided Replacement Mirrors (three)
- 7 – Rotor Exchange Tool

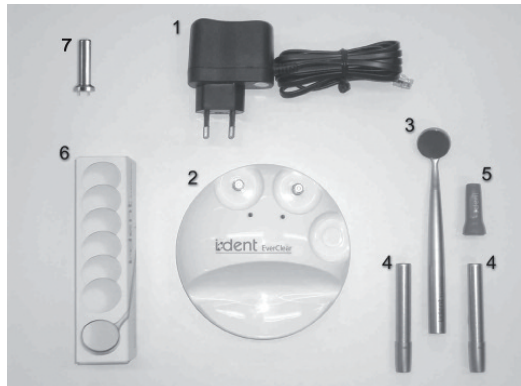
Illustration 1

3.2 Connecting the power supply to the battery charger

Plug the phone-jack type power cord connector (1) into the socket on the underside of the battery charger (2) (Illustration 2).

Illustration 2

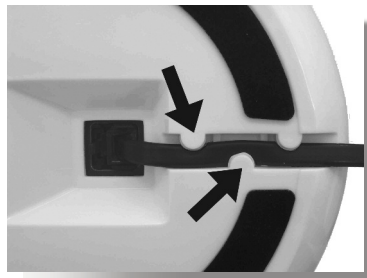
Illustration 3

Thread the cord underneath three clips provided adjacent to the socket (Illustration 3). The cord fits snugly under the clips to prevent tension on the phone-jack.

CAUTION: Place charger unit on a stable, horizontal surface at least 1.5 m from the patient!

3.3 Charging the batteries

Fully charge both batteries prior to first use. Place battery (4) onto the charging connector (Illustration 4).

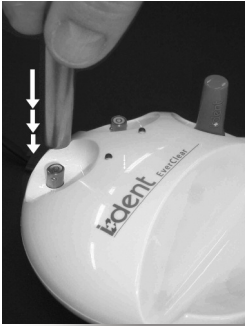
Illustration 4

Illustration 5

Once you encounter resistance, push on the battery to lock it in place (Illustration 5). Green light will turn on, indicating charging is in progress. When the light turns off, charge is complete. When not in use, batteries should be kept on charger to compensate for self-discharge and to ensure maximum battery operating time. These batteries are memory-free and should be kept charging when possible. Depleted batteries take slightly more than one hour to charge fully. A fully charged battery will power The EverClear™ for approximately 45 minutes. As is typical, battery capacity reduces over the course of a battery's lifetime. Replacement is at user's discretion.

NOTE: These batteries may only be charged using the provided I-DENT battery charger.

NOTE: Do not insert metallic parts into open end of battery; Short circuit could occur.

CAUTION: Never autoclave batteries !

4. INSTRUCTIONS FOR SAFE OPERATION

Illustration 6

Illustration 7

Gently insert battery (4) into the EverClear™ handle (3). You will feel the connector guide the battery automatically into the **OFF** position (Illustration 6). Once you feel resistance, push the battery all the way into the handle (Illustration 7). A click will be heard. To turn EverClear™ **ON**, rotate the battery a quarter turn in either direction. To stop EverClear™, rotate the battery a quarter turn in either direction. In the **ON** position, the EverClear™ mirror will rotate to repel water and debris.

EverClear™ improves the efficiency of dental procedures by remaining free of water droplets and debris and eliminating the need for mirror removal from the mouth for the purpose of cleaning.

Due to electrostatic properties of certain composite materials when drilled, these may form slightly visible rings on the mirror surface. Any build-up can immediately be removed by momentarily directing spray at the mirror. Visibility can be further optimised, by slightly increasing the volume of water spray from the handpiece.

In general, EverClear™ eliminates the need to reduce spray volume, which is often done by practitioners in order to maintain visibility. Increased spray volume provides improved cooling for the tooth, which is a significant advantage of EverClear™.

The mirror rotation will slow or stop safely upon contact with any object. While contact with intra-oral tissues is completely safe, you should avoid contacting the rotating surface with suction tips, soft tissue or the handpiece in order to maintain continuous operation. If rotation stops due to such contact, it will resume shortly after the mirror surface disengages the object which stopped its rotation.

NOTE: In order to reduce scratching of the mirror surface during operation try to keep the mirror away from the head of the high-speed or contra-angle. Also, avoid any contact of the EverClear™ with rotating burs to prevent damage to the surface finish of the mirror or the instrument.

NOTE: It is normal for the EverClear™ to feel slightly warm during operation. It will feel warmer, if resistance to rotation as described above, is encountered.

When a procedure is completed or interrupted, turn EverClear™ to **OFF**. If the battery becomes depleted prior to completing a procedure EverClear™ will stop and the depleted battery must be replaced with the second, charged battery.

NOTE: There will be no reduction in speed due to diminished battery charge. Speed will be maintained until the mirror stops due to battery depletion.

NOTE: Keep battery contacts dry for proper operation.

4.1 Mirror exchange

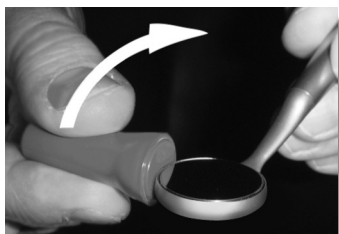
Illustration 8

Illustration 9

To remove the mirror from its support seat (for exchange if scratched or for cleaning and service), approach with the provided magnetic Mirror Exchange Tool (5) from the side until it touches the housing (Illustration 8), then simply flip the tool to the top of the mirror surface (Illustration 9).

NOTE: The Mirror Exchange Tool may never impact the mirror surface, for example by approaching directly from the top. Doing so involves always the risk, that the important magnetic forces, which build-up between the tool and the mirror, may provoke some slipping of the tool through the fingers and thus result in an impact which could damage the bearings.

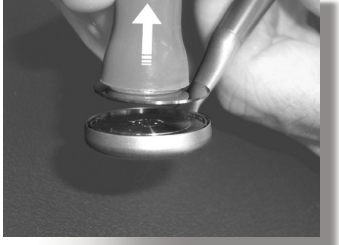
Illustration 10

Illustration 11

Pull the tool up. The mirror will simply lift from its supporting seat (Illustration 10). Using minimal pressure, carefully clean the mirror support seat with alcohol and a soft, lint free cloth. Grasp the mirror by its edge and slide it off of the tool (Illustration 11). Clean both sides of the mirror with alcohol and a lint-free cloth. Then reposition the magnetic tool on top of the unused side of the mirror.

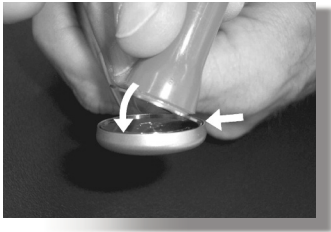
Illustration 12

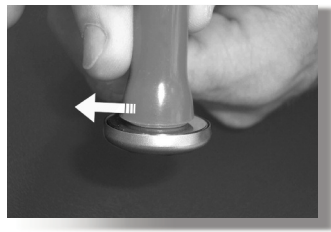
Illustration 13

Slant the mirror against the support surface (Illustration 12). Shift it laterally until it meets the perimeter of its seat on the turntable, lowering it down at the same time. The mirror will mate the curvature of the centering seat as it lowers onto the turntable. Slide the magnetic Mirror Exchange Tool laterally off the seated mirror (Illustration 13). **Make sure the mirror is centered and fully seated.**

ATTENTION: After each mirror change, activate the motor to ensure there is no vibration or incorrect mirror positioning. Test the EverClear™ before it is used with a patient. Incorrect positioning could lead to the mirror dislodging from the rotor when activated.

To maintain brilliant clarity when both sides of the mirror become scratched, use a new mirror. See chapter 9, Spare Part, to order.

5. HYGIENIC RECONDITIONING AND STERILIZATION

For intense build-up of contamination, cleaning the EverClear™ under running tap water is recommended. Following this, or when precleaning is not required, clean the outer surface of the EverClear™ by wiping it with disinfectant, using approved instrument disinfectants with an alcohol base. **If not yet done, please remove the battery now.**

Remove the mirror from its support seat as described in chapter 4.1. In doing so, approach the mirror always laterally with the magnetic Mirror Exchange Tool. Make sure the tool does not impact the mirror to avoid shock to the ball bearings. Using minimal pressure, carefully clean the mirror support surface with alcohol and a soft, lint-free cloth. Clean both sides of the mirror in the same manner.

NOTE: The mirror support surface and the mirror can also be cleaned and disinfected using instrument disinfectant with an alcohol base. In this case, however, repeat cleaning with alcohol should be carried out in order to remove residual disinfectant.

The area under the mirror support seat should also be cleaned regularly (with normal use we recommend weekly) or when build-up is excessive. To do this you have to remove the Rotor first.

5.1 Removal of the Rotor assembly

To remove the Rotor, place both pins of the Rotor Tool (7) into the two small openings in the center of the rotor (Illustration 14). Rotate the tool slightly until the pin can be heard to engage and the mechanism rests flush against the mirror support surface (Illustration 15).

The rotor bearing fitting will loosen by rotating the tool counter-clockwise using minimal axial force. After approximately seven rotations the rotor will loosen entirely from the head housing and will remain magnetically fastened to the Rotor Tool.

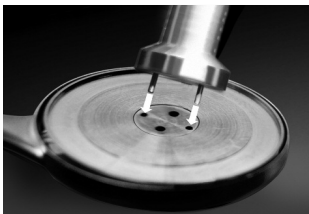
Illustration 14

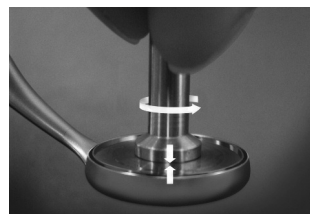
Illustration 15

Clean the underside of the rotor with alcohol and a soft, lint free cloth (Illustration 16). Take care not to allow particles to drop into the sensitive bearing. With a Q-tip soaked in alcohol or in a disinfectant with an alcohol base gently wipe off the internal surfaces and and rinse any possible disinfectant residue with clean tap water.

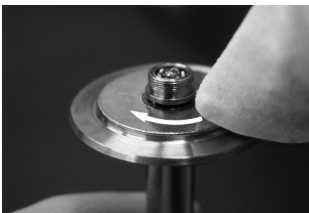
Illustration 16

Illustration 17

After cleaning and drying the rotor, insert it into the head housing again with the tool, being careful that the fine threads of the bearing fitting engage smoothly and are not skew (Illustration 17). Screw the rotor down until it bottoms out. Then pull the Rotor Tool up and off.

If re-lubrication is necessary, do it at this time, as described in Chapter 6.

Reposition the cleaned mirror as described in Chapter 4.1 back on the mirror support surface. Place the instrument, **without the battery**, into a sterilization bag and autoclave it according to the instructions from the sterilizer manufacturer. The maximum nominal temperature for sterilization is 135 °C (= 275 °F).

CAUTION: Do not submerge the EverClear™ into an ultrasonic or a cold sterilization bath.

CAUTION: Do not use a dry heat sterilizer, Chemiclave® type sterilizer or a dental Washer Disinfector (Thermal Disinfector).

CAUTION: Please do not autoclave the battery!

NOTE: Do not forget to carry out regular lubrication as detailed in chapter 6.

ATTENTION: After each mirror change, activate the motor to ensure there is no vibration or incorrect mirror positioning. Test the EverClear™ before it is used with a patient. Incorrect positioning could lead to the mirror dislodging from the rotor when activated.

5.2 Battery and other surfaces

The outside of the battery can be cleaned and disinfected by wiping it with disinfectant. Use only approved instrument disinfectant with an alcohol base for this purpose. The battery contacts must remain dry.

The surface of the charger device can also be cleaned and disinfected using a mild cleaner or disinfectant with an alcohol base. The same is also true for the mirror- and rotor exchange tools. To avoid scratches, always use a soft, lint-free cloth to clean plastic surfaces

NOTE: Do not sterilize the Mirror Exchange Tool.

6. MAINTENANCE

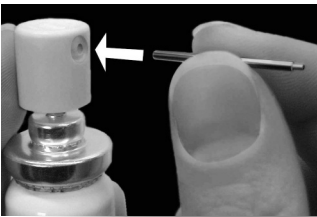
Illustration 18

EverClear™ bearings only require lubrication once per month or if bearing noise increases. Please use only original I-DENT “Long-life“ maintenance spray (EverClear™ accessory part ID50-3020). This spray contains a special, “Long-life“ lubricant that makes extended maintenance intervals possible.

Spray can preparation: Prior to initial use, guide the blunt end of the supplied nozzle extension into the opening of the spray head until it bottoms, about 4 – 5 mm deep (Illustration 18). Spray a sample of lubricant onto a paper towel once or twice. This ensures that the dosing mechanism is completely filled.

To lubricate, remove the mirror from the rotor by using the magnetic mirror exchange tool (as described in 4.1). Guide the fine nozzle end of the spray can into one of the two larger opening in the center of the rotor until it bottoms, then press the nozzle head of the spray can **only one time** (Illustrations 19 and 20). An exact quantity of maintenance spray is applied directly to the bearings in this way.

Carefully wipe the lubricant off the surface of the rotor. Place the mirror back on the rotor and run EverClear™ for one minute.

Illustration 19

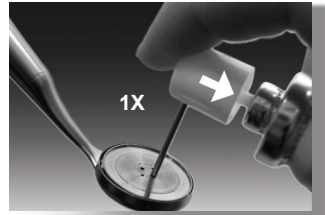
Illustration 20

NOTE: Do not repeat application. Excess lubrication may temporarily impair performance.

7. GENERAL CARE

As with all dental instruments, cleanliness is paramount. In particular, make sure that the mirror seat below the mirror surface is kept clean and particle free. All EverClear™ surfaces, charger, mirror exchange tool, handpiece, battery and mirror may be cleaned with detergent, alcohol or alcohol-based disinfectants. Dampen a clean, lint free cloth with cleaner/disinfectant and thoroughly wipe external surfaces. Battery contacts do not require cleaning and must be kept dry.

8. TROUBLESHOOTING

In rare circumstances mirror rotation will not resume after a stoppage. This indicates increased friction in the rotor's bearings, which might be due to depletion of lubricant after long usage, or to deposits forming on the bearings, possibly due to autoclaving if the steam atmosphere contains contaminants. First aid: place your finger on top of the mirror and spin it clockwise. Rotation will restart. At next maintenance, or prior to next use perform lubrication as described above. If lubrication does not eliminate the effect, deposits are the probable cause and the instrument should be returned to your dealer for service.

CAUTION: Batteries may never be autoclaved. Autoclaving the EverClear™ with battery inserted may result in damage to the instrument.


Accidentally autoclaving or rapidly discharging the battery via a short circuit may lead to leakage of a small amount of battery fluid at the connector. As this fluid is aggressive, it must be removed immediately. To do this, please unscrew the handpiece prolongation tube and carefully clean all parts, as well as your hands, using rinse water. The battery must be discarded according to local regulations.

In case of problems, please contact your local dealer or e-mail I-DENT Innovations for Dentistry: info@i-dent-dental.com

9. SPARE PARTS AND CONSUMABLES

	Order No.
Always use only approved or original replacement parts:	
Double-sided, precision replacement mirrors (Six mirrors/12 usable surfaces)	ID50-2070
Battery (Metal encapsulated NiMH with quick coupler)	ID50-2060
“Long life” maintenance spray (Ball bearing lubricant for topping-up)	ID50-3020
Mirror Exchange Tool (Magnetic tool for easy exchange of mirrors)	ID40-0010
Rotor Exchange Tool (Special wrench allowing for rapid rotor exchange)	ID40-0021
Replacement Rotor (Screwable rotor-element incl. double ball bearing)	ID20-0130

10. STORAGE AND TRANSPORT CONDITIONS

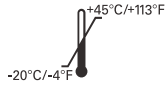11. EXPLANATION OF SYMBOLS USED ON THE DEVICE

The EverClear™ Denal Mirror is labeled with the CE mark in compliance with the regulations of the directive 93/42/EEC.

This symbol indicates that a product or instrument (here, the EverClear™ Dental Mirror) can be sterilized at the indicated temperature (135° C = 275° F).

Attention: Please see the documentation that accompanies this product!

Please see the Operating Manual!

This symbol refers to the special disposal of electrical and electronic devices in EU countries. Please do not discard this device in household garbage. Check the proper means of disposal in your country at your community recycling or waste center or at your dealer. Take care to dispose of properly.

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